

The strip-free Accu-Chek® Mobile system: A new way of testing with 50 tests on a continuous tape.

▶▶ Just test. Let the meter do the rest.




Measuring



Open tip cover. Slide completely to the left until meter switches on.



Press prime and lance button all the way down to prime. Press down again to obtain blood sample.

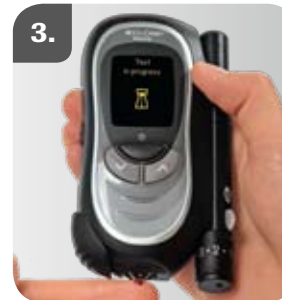

Immediately apply blood in the middle of yellow test area at cassette tip. Obtain result after 5 seconds.



Close tip cover completely. Used test is safely stored in the cassette. System turns off automatically after 15 seconds.

Changing lancet



Advance to next lancet by pushing lancet changer back and forth once. Number of remaining lancets is displayed in lancet counter.

Replacing test cassette – after using all 50 tests:



1. Open tip cover. Slide completely to the left until meter switches on.



2. Slide side button down to open cassette door.



3. Remove used cassette and insert new cassette.



4. Close cassette door first, then shut tip cover.

Replacing lancet drum – after using all 6 lancets:



1. Take off cap.



2. Remove used lancet drum.



3. Insert new drum. Used lancet drum shows red stripe and cannot be re-used.



4. Put cap back on.